



The ALD Zone

Helping Professionals Provide Successful Solutions with ALDs since 1993

Father's Day
is around the
corner.

Some gift ideas:



- TV Listening System
- Shake Awake Smoke Detector for RV or vacation home.
- Amplified Cordless Phone.

HLAA Conference 6/18-20

Becky will be attending. Let her know if you are going, too.

Welcome to the new Beyond Hearing Aids' Newsletter, newly named The ALD Zone!

New Name. Same Benefits. Fast – Easy to Read – Easy to Stay Up-to-Date.

We've been extremely busy. You should receive an email from us announcing the updated www.BeyondHearingAids.com website. Now you can find anything you want to know about a product from one place. No more researching data. Find product manuals, technical specifications and videos on many products in the store. More are being added weekly.

More information in one place for better education!

What more could you want?

What? You have ideas on what would make your life easier with ALDs?

Email me at Beckym@beyondhearingaids.com

Lifetone HL Fire Alarm and Clock with Bed Shaker

Here is a new smoke detection system/alarm clock you may want to know about. It uses a patented sensor to hear your existing T3 smoke alarm (see pink box on page 2). It then emits the low and multiple frequency 520 Hz square-wave sound pattern and vibrates the bed shaker to awaken the user to the smoke alarm. The Lifetone HL™ Fire Alarm and Clock functions as a standard alarm clock with bed shaker and loud 85 dB alarm, and is designed to awaken heavy sleepers, seniors, children and people with hearing loss when the smoke alarm sounds. Works only with UL-approved T3 smoke alarms (see pink box on page 2). The Lifetone Fire Alarm and Clock is also available with a T3 smoke detector (battery operated), HC-LIFETONE/KIT. Additional T3 battery operated smoke detectors are available for \$8.95 each (HC-SMOKE/BATT)

Research shows that 95% of respondents with hearing loss woke to the square-wave alarm, 39 percentage points better than current smoke alarms.

Two year warranty.

A downside to this alerting system is that it relies on a sound monitor to hear the smoke detector. Excessive noise levels (70dB or higher) may obstruct the emergency signals given by your smoke alarm to the Lifetone HL Fire Alarm, which may result in personal injury or death. There is also a warning not to use this system with any acoustic sound-dampening wall or ceiling materials. These materials may reduce sound levels and obstruct the emergency signal given by the product, which may result in personal injury or death.

Other Features: Extra large clock display; 7 day battery backup; shows AC power loss and low/no battery; shows unplugged or improperly connected bed shaker (included); detects T3 alarms through walls, closed doors and multi-story houses. Dimensions: 8-1/2" L x 5" W x 4" H



**Lifetone HL Fire Alarm and Clock
HC-LIFETONE
\$179.95**

**Lifetone HL Fire Alarm and Clock
with T3 Smoke Detector
HC-LIFETONE/KIT
\$184.95**



Beyond Hearing Aids, Inc.
463 Erlanger Rd., Ste 1, Erlanger, KY 41018
www.BeyondHearingAids.com 800-838-1649

**Helping Professionals Provide Successful Solutions
with ALDs since 1993**

New BeyondHearingAids Website Preview:

Yes, we're busy getting the final touches on the new website with more support details added every day:

- All new even easier navigation
- 5 captioned welcome messages
- 35+ Products with captioned videos
- 30+ Products with Product Detail Sheets
- 30+ Products with Technical Specifications
- 75+ User Manuals

You don't need to Google to research ALDs. You can find it all here from The ALD Specialists that Professionals Trust.

ALD News:

What is a T3 smoke alarm pattern?

Otherwise known as the "three pulse" or "temporal three" pattern, the T3 sound pattern is three half-second repetitions of the same sound separated by half-second pauses, followed by a 1.5 second pause after the third repetition. The alarm sounds like this: "beep beep beep pause", "beep beep beep pause", "beep beep beep pause". (Note that if your current smoke alarm does not use this pattern, it will not be compatible with Lifetone products.) The T3 pattern has been used as a standard acoustic pattern in smoke alarms to indicate an immediate need to evacuate.

Discontinued Items:

GE-PHONE1 – Big Button Cordless Phone
TD-BEBE – Portable Video/Sound Baby Monitor
HC-BBUTTON – Big Button Corded Phone

It's Good to Go Green

We've been recyclers here at Beyond Hearing Aids for a while now but I have to admit I was the last in the group to give up water bottles. I've been using a BPA free container and don't feel guilty (or thirsty) anymore.

I thought it an appropriate time to ask you to consider how you receive this newsletter. The email subscription sends you an email with a link that takes you to the newsletter page. You can read and print if needed.

The mail subscription mails you a hard copy. Some people prefer to read on the run or don't like reading newsletters online. You need to do what works right for you.

We'd like to update our subscriber options. Can you take one minute (I've timed it) to confirm your options: Email, Mail, or Both.

Please reply to custserv@beyondhearingaids.com or phone in 800-838-1649 and leave a message with your name and preference: EMAIL, MAIL or BOTH.

We'll update your subscription immediately.

Don't Delay – Do It Right Now!!